AED remote monitoring system

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
MOH strongly recommended maintenance of AED including batteries & pads

AED Remote Monitoring System

AED remote maintenance system (AED Linkage) can provide you easy maintenance of AED including pads and battery controlled by wireless modem.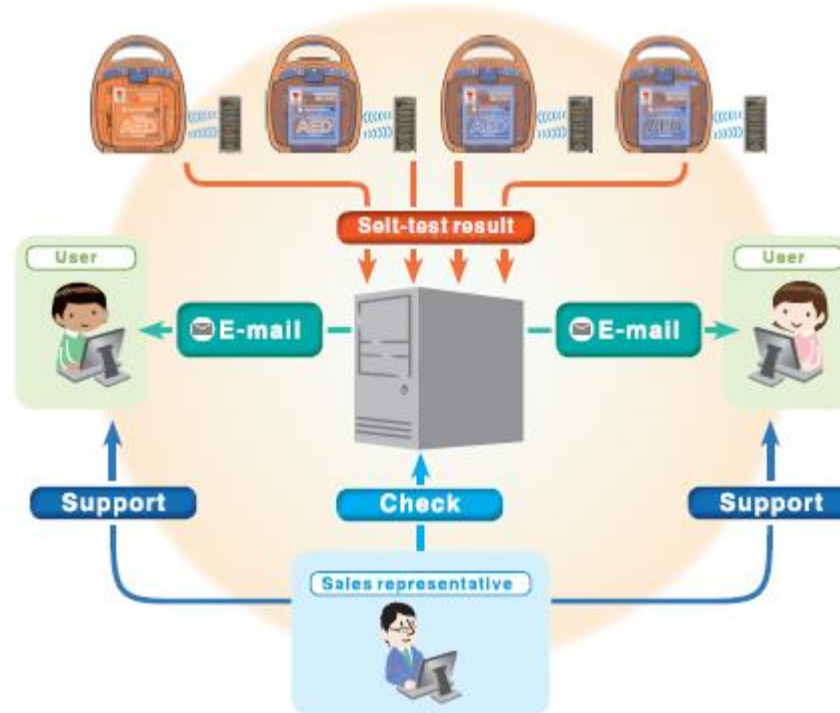

Difficulties of AED maintenance

- Expiration dates management
- Identification of error types
- Cost to visit to installation site

Difficulties of AED maintenance

- **Expiration dates management**
- Identification of error types
- Cost to visit to installation site


Expiration dates management

- Result of self-test

- AED status
- Expiration date of defibrillation pads
- Expiration date of battery
- Remaining power of battery (%)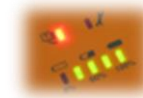

- Software version, Battery lot number
- Status of connection to server

Difficulties of AED maintenance

- Expiration dates management
- **Identification of error types**
- Cost to visit to installation site

E-mail – sample

AED notification
Nihon Kohden AED Linkage System
宛先:

[1 / 1]

Date: 2015-01-05 22:15:01

Subject: AED notification AED-2100K Location: ~~Resubok~~ Office Serial No.:000001

Content:

All AEDs at [NIHON KOHDEN ~~EUROPE SUBGERMAN SALES DIVISION Resubok Office~~] are in operating condition.(001)

----- Self Test Report -----

Date: 2015/ 1/ 5

User: NIHON KOHDEN ~~EUROPE SUBGERMAN SALES DIVISION~~

Facility: ~~Resubok~~ Office AED-2100K

Serial number: 00454

Overall test result:Normal

Pad expiration date:2015/ 6

Pad test result:Normal

Battery power:86% :

Battery expiration date:2019/ 9/24

Battery test result:Normal

Battery model:NKPB-1430K

Lot : 00624020

Abnormalities:Nothing

AED box door:Close

Nihon Kohden AED Linkage System

<https://oversea.arm.nihonkohden.co.jp>

- ✓ Self-test result
- ✓ Local language
- ✓ Any E-mail address is available

E-mail – receiving time


Communicate AED ⇔ Terminal	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
E-mail when AED has problem					✓	✓	✓	✓	✓	✓
E-mail when AED is normal					✓					✓

Difficulties of AED maintenance


- Expiration dates management
- Identification of error types
- **Cost to visit to installation site**

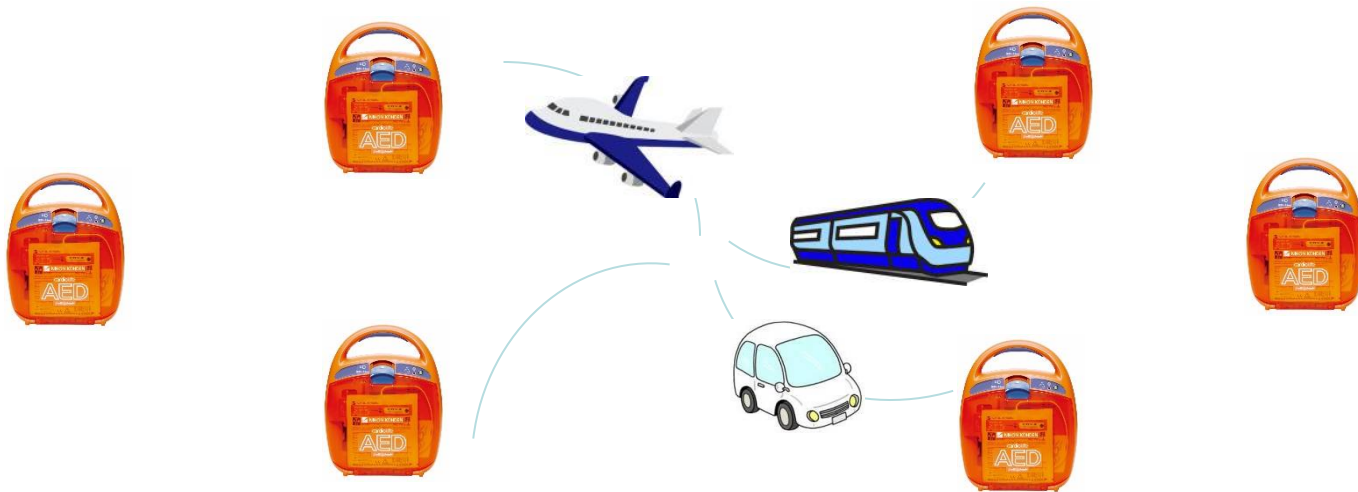Saving total maintenance cost

Easy maintenance by AED Linkage!!

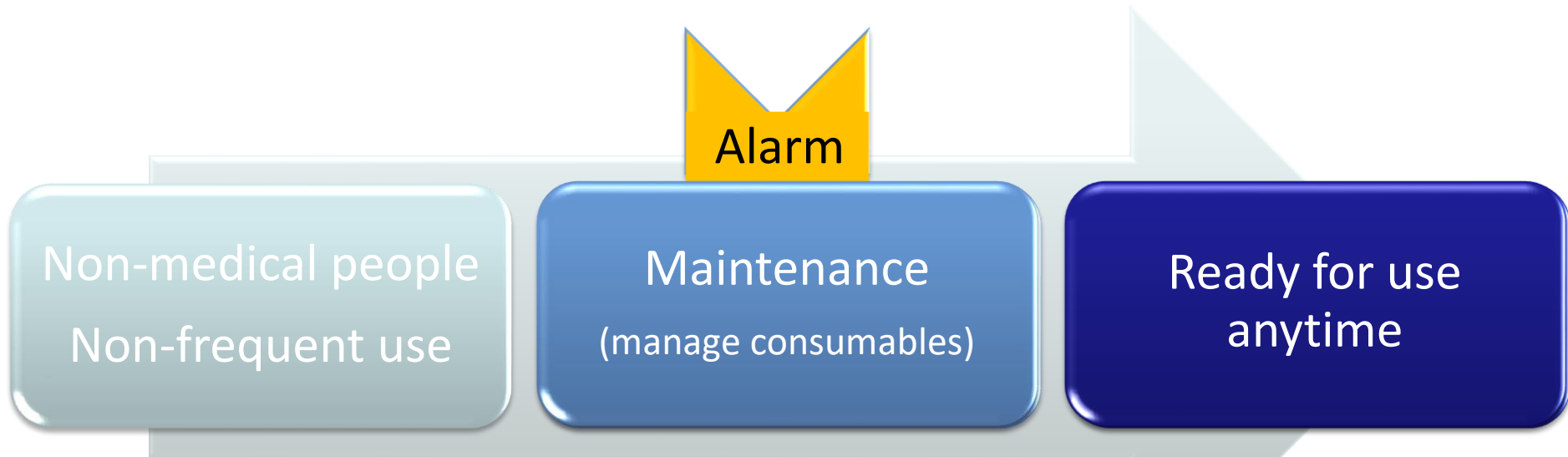***Support your daily maintenance
to make your **AED** ready to use in anytime!***

- Checks that the AED is ready to use
- Checks the battery
- Checks the pad expiration date
- Checks different AEDs in different locations

- Thank you -